


# Attends® Soft



## Fitting Guide



**1** Remove the adhesive fixation tape on the back of the pad.

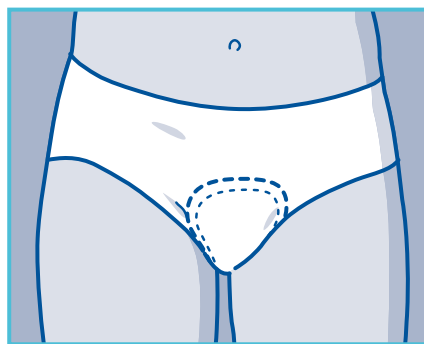

**2** Place the pad securely in your own close fitting underwear ensuring that the fixation tape is away from the skin.

## Hints and Tips

### Selection

- Choose the absorbency of pad required according to individual client assessment.
- For optimal performance, we suggest using the product in combination with close fitting underwear.
- The smaller the pad chosen, the greater the dignity and comfort for the user.

### Disposal

- Roll up pads and dispose as per your Trust's clinical waste policy.
- Attends Soft 1 - 3+, you may use the single wrap to dispose of the product.
- Do not flush the pad down the toilet.



Attends Soft is tested and approved by proDERM, Institute for Applied Dermatological Research.

For more information visit [www.attends.co.uk](http://www.attends.co.uk)